


Using the VivaLNK Temp Patch

1 PREPARE AND PLACE PATCH

- Remove the patch from the cradle.
- Remove the backing from the side marked "1" of an adhesive strip. Align the hole in the adhesive with the metal temperature probe on the patch. Adhere the adhesive to the patch.
- Remove the backing from the side marked "2" of the prepared patch. Adhere the patch under your arm, with the probe side inside the armpit. The adhesion area should be free of body hair and products such as lotion or deodorant.

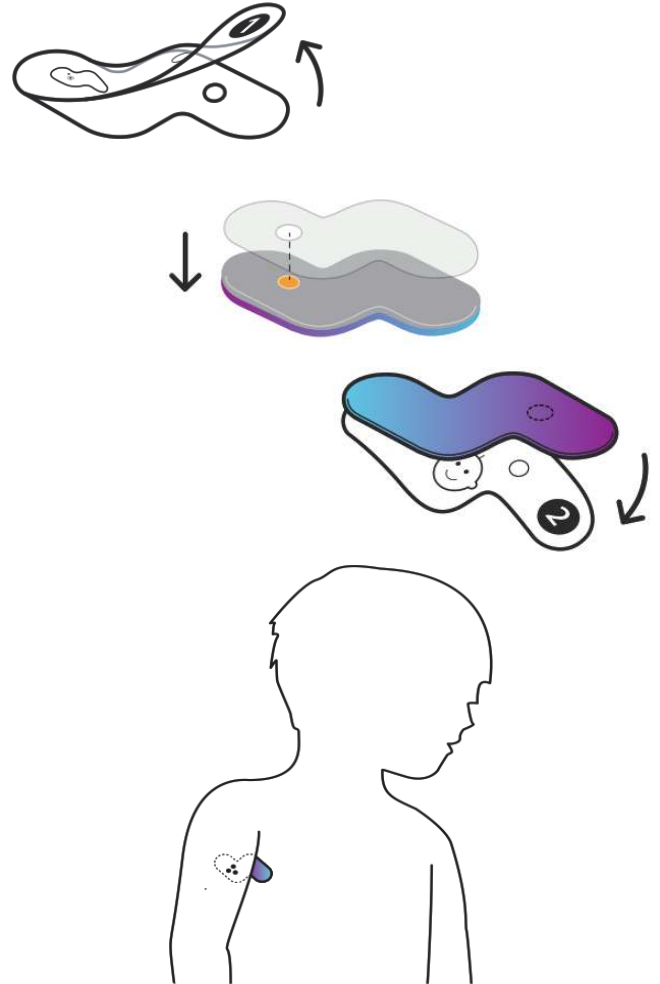

2 PAIR THE PATCH (First use only!)

- In the VitalCare app on the patient tablet, click "More" and then select "Installed Devices" from the menu.
- Select "Add Device" from the menu.
- Select the "Temperature Patch" option. You should see a lighthouse animation as the tablet searches for nearby devices.
- Choose the correct patch from the list. You should see a "SUCCESS" message indicating proper pairing.

3 USING THE PATCH

- Readings should appear once per hour while the tablet is turned on and in range of the patch.
- The patch can be temporarily removed for bathing, and the adhesive should be changed as needed.

Things to know:

- It will take 8-10 minutes for the patch to come up to body temp and provide accurate readings.
- The axillary (armpit) temperature is lower than the oral temperature. This is normal and expected.
- The patch needs to be charged roughly every 5-7 days.